

1. PRODUCT

Product Designation

Face mask with protective flap, earloops and 6 layers. Level 2.

Reference
HSMC019

EAN
-

Product Description


Textile mask intended to be used by the community in general, as a preventive protection equipment.



- Reusable mask (10 uses);
- Three ply mask;
- With earloops;
- With adjustable nose clip;
- Six-layer protective structure;
- Antibacterial finish;
- Adult and child sizes available;
- Available in multiple colors.

Face masks help to prevent particles expelled by the user (e.g. saliva, mucus) from reaching other people or surfaces. This prevents the spread of the new Coronavirus (SARS-CoV-2), as well as protects the user from particles or fluids from others.

2. CHARACTERISTICS

Type	Community mask, 6 layers, level 2.
Composition	4 layers – Fabric 100% Pes 95 g/m ² 1 layer – Non-woven fabric 100% Pes 73 g/m ² 1 layer – Pellicle 100% PU
Instructions for Use	<p>It is recommended to wash the mask before the first use.</p> <p>Washing Complete washing cycle of, at least, 30 minutes at 60 °C with detergent. Use mask only after being well dried.</p> <p>Sterilization (if necessary) After washing, at a maximum temperature of 134 °C, for 3 minutes.</p> <p>Mask Placement</p> 

1. Wash your hands with soap and water or an alcohol-based solution **before** putting on the mask;
2. Place the mask over the face in the correct position (folding edge up and colored part out), making sure that there are no spaces between the face and the mask;
3. Hold the mask by the supports / elastic bands and adapt to each ear;
4. Adjust the mask next to the nose and chin, without touching the face of the mask.

Mask Removal




1. Wash your hands with soap and water or an alcohol-based solution **before** removing on the mask;
2. Remove the mask just by holding the supports / elastic bands;
3. Keep the mask away from the face and clothing, to avoid touching potentially contaminated surfaces of the mask;
4. Place the mask in the appropriate container (individualized in a closed plastic bag, until it is placed in the washing machine) and sanitize your hands again, with soap and water or an alcohol-based solution.

Approvals

TESTS	RESULTS	INITIAL	AFTER 10 WASHES
	BFE Bacterial filtration efficiency >90%		99,8%
Bioburden Microbiological cleaning <=30cfu/g		<1	<4
SPLASH RESISTANCE 120 mmHg (corresponds to the mean systolic blood pressure)		32 – tests 0 – NOK	32 – tests 0 – NOK

Measurements

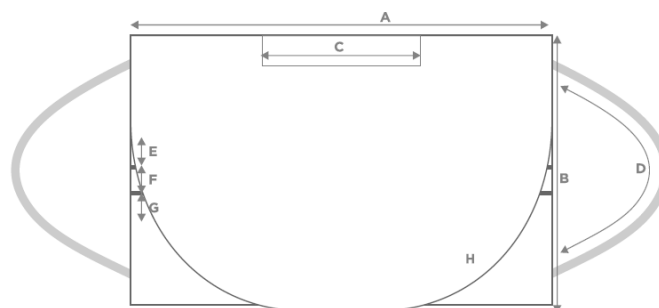

A	WIDTH	17CM
B	HEIGHT	10,5CM
C	ADJUSTMENT	8,5CM
E/F/G	PLEATS	1CM
D	ELASTIC BAND	18CM
H	PROTECTIVE FLAP	

3. PACKAGING

Product sold by the unit or in packs of 10 units.

Note: These specifications are true and accurate according to the data provided by our supplier.