

1. PRODUCT

Product Designation

Reusable social mask Level 3 (Pack of 2 units) White / Gray

Reference
HSMC020

EAN


Product Description



- Reusable mask;
- With earloops;
- With flexible nose clip;
- Three-layer protective structure;
- Antibacterial and antistatic finish;
- CITEVE certified (Test Report No. 4830/2020)
- Available in white or gray.



2. CHARACTERISTICS

Type	Social mask, 3 layers, level 3 (adult).
Composition	<p>Protective structure with 3 layers:</p> <ul style="list-style-type: none"> • External and internal layer – Circular mesh 100% polyester. Antibacterial and antistatic finish. Density: 130 g/m² (±5%). • Inner layer – Non-woven filter 100% Polyester. Density: 51 g/m² (±5%). EN 29073.
Functionality	<p>This item meets the standard performance requirements, in accordance with the standard applicable to Community Masks.</p> <p>“Masks intended for use within the scope of COVID-19”:</p> <ul style="list-style-type: none"> • Air Permeability / EN ISO 9237:1995 – Average air flow = 26,0 l/min (minimum requirement: ≥ 8 l / min) / Report 4160/2020 • Particle Retention Capacity (PRC) / MI 142/00- CITEVE / - PRC = 82% (minimum requirement: ≥ 70%) • Domestic Washing and Drying Behaviour / Standard: ISO 6330:2012
Washing Instructions	<p>The masks should be washed using a normal, complete washing cycle at 60 ° C with detergent, after each use. The user must ensure that the mask is dry before use. It does not allow dry cleaning.</p> <div style="text-align: center;">  </div>
Mask Lifetime	30 washing cycles according to the recommended indications. Mask certified by CITEVE for 5 washes (Test report No. 4830/2020).

**Instructions
for Use**


This mask should be used as a complement to the recommended protection and hygiene measures and to the rules of social distance fundamental for the control of COVID-19. For more information, go to <https://covid19.min-saude.pt>.

The reusable mask must be handled correctly. The following instruction are recommended by the DGS guidance (nº 019/2020 of 03/04/2020):


- a) The mask must be washed before its first use, according to the recommendations for conservation and cleaning;
- b) Wash your hands, with soap and water or an alcohol-based solution, before putting on the mask;
- c) Putting on the mask with the inner layer facing the face, and the outer layer facing out;
- d) Adjust the rigid end of the mask, covering the mouth, nose and chin with the mask, making sure that there are no spaces between the face and the mask;
- e) The mask should not be touched while it is in use, if it happens, wash your hands immediately;
- f) Maximum period of use of each mask: 4 hours per day. The mask must be replaced with a new one at the end of that period or as soon as it is wet. The removed mask must be washed before being reused;
- g) If you accidentally get the mask wet, replace it with a dry one as soon as possible. The wet mask must be washed before being reused;
- h) The mask must be removed from the back (not touching the front of the mask), holding the ties or elastics;
- i) The mask must be placed individually in a closed plastic bag, until it is placed in the washing machine;
- j) Wash your hands at the end of the use and handling of the mask.

- The protection of the mask is only guaranteed if you follow the cleaning and maintenance instructions.
- The mask must be stored in a dry place and protected from potential contamination.
- In case of defect or damage do not use the mask, replace it with another one.
- At the end of the mask's lifetime, wash it under the conditions defined in "Washing Instructions" and after washing it must be discarded as non-hazardous textile waste. If you choose not to wash it, dispose of it as biological waste.

Mask Placement Scheme

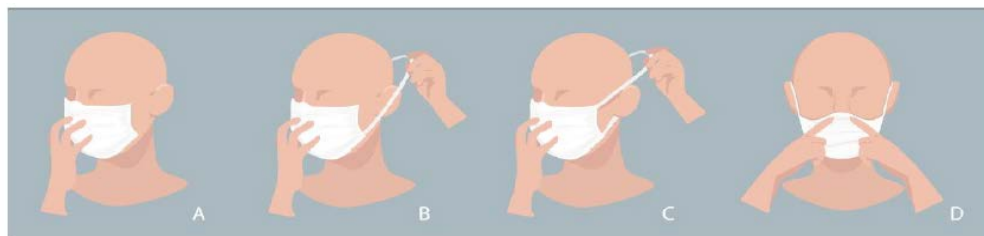

Mask Removal Scheme

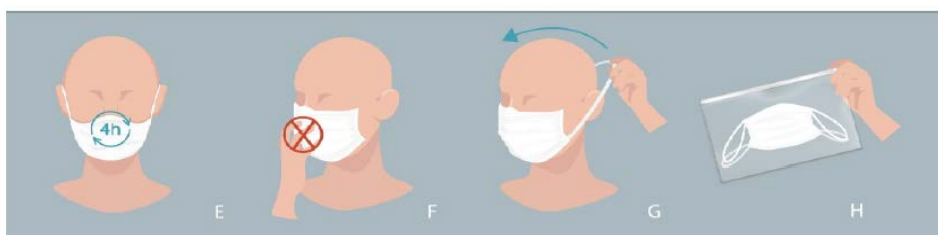

Usage Limitations

This device is not a medical device. It does not comply with the regulation of surgical masks (Regulation EU/2017/745) or of personal protective equipment (Regulation EU/2016/425). The mask should be used completely dry after washing. The wet/damp mask decreases the protection of the mask.

The mask can only be used by someone else after washing.

At the end of the recommended daily use or whenever you return home, the mask must be washed before being reused.

3. PACKAGING

Product sold in packs of 2 units, properly packed in a reusable sealed bag to store the masks when not in use. Inside the bag there is an explanatory leaflet with all the rules and instructions that the product requires to present.

Note: These specifications are true and accurate according to the data provided by our supplier.