

## 1. PRODUCT

### Product Designation

Mask with transparent visor Level 2

**Reference**  
HSMC021

**EAN**

### Product Description

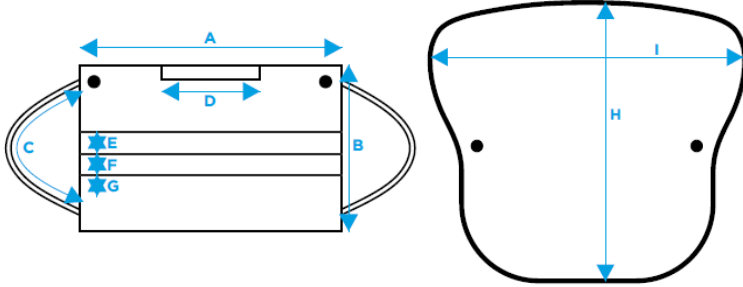

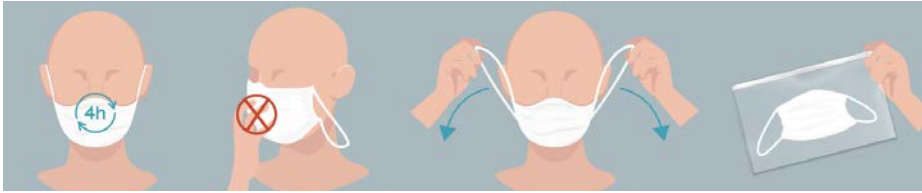


- Reusable mask (10 uses);
- With removable visor;
- Three ply mask;
- With earloops;
- With adjustable nose clip
- Three-layer protective structure plus visor;
- Antibacterial finish;
- Reusable visor.

Face masks help to prevent particles expelled by the user (e.g. saliva, mucus) from reaching other people or surfaces. This prevents the spread of the new Coronavirus (SARS-CoV-2), as well as protects the user from particles or fluids from others.

## 2. CHARACTERISTICS

<b>Type</b>	Community mask with visor, 3 layers, level 2.
<b>Composition</b>	<p><u>MASK</u></p> <ul style="list-style-type: none"> <li>• 2 outer layers - Fabric 100% Pes 95 g/m<sup>2</sup></li> <li>• Inner layer – Non-woven 100% Pes 73 g/m<sup>2</sup></li> </ul> <p><u>TRANSPARENT VISOR</u></p> <ul style="list-style-type: none"> <li>• Material - PETG</li> </ul>
<b>Washing Instructions</b>	<p><u>MASK</u></p> <p>Washing before first use is optional.</p> <p>After use, they must be washed using a complete wash cycle of at least 30 minutes, at 60 °C with detergent. They can only be used after they are completely dried.</p> <p>If necessary, the masks can be sterilized, after washing, at a maximum temperature of 134 °C, for 3 minutes.</p> <p><u>TRANSPARENT VISOR</u></p> <p>Clean manually using a clean cloth, water and soap.</p>
<b>Elimination</b>	When the transparent visor is worn out and does not allow its use, it must be placed in the plastic recycling bin.
<b>Risks</b>	No risks associated with this product are identified.

<p><b>Approvals</b></p>	<p>Level 2 mask, according to the document “Masks intended for use within the scope of COVID-19”, of April 14, 2020.</p> <table border="1"> <thead> <tr> <th>TESTS \ RESULTS</th> <th>INITIAL</th> <th>AFTER 10 WASHES</th> </tr> </thead> <tbody> <tr> <td><b>BFE</b> Bacterial filtration efficiency &gt;90%</td> <td>99,8%</td> <td>94,8%</td> </tr> <tr> <td><b>Bioburden</b> Microbiological cleaning &lt;=30cfu/g</td> <td>&lt;1</td> <td>&lt;4</td> </tr> </tbody> </table>	TESTS \ RESULTS	INITIAL	AFTER 10 WASHES	<b>BFE</b> Bacterial filtration efficiency >90%	99,8%	94,8%	<b>Bioburden</b> Microbiological cleaning <=30cfu/g	<1	<4												
TESTS \ RESULTS	INITIAL	AFTER 10 WASHES																				
<b>BFE</b> Bacterial filtration efficiency >90%	99,8%	94,8%																				
<b>Bioburden</b> Microbiological cleaning <=30cfu/g	<1	<4																				
<p><b>Measurements</b></p>	 <table border="1"> <tbody> <tr> <td><b>A</b></td> <td>WIDTH</td> <td>17CM</td> </tr> <tr> <td><b>B</b></td> <td>HEIGHT</td> <td>10CM</td> </tr> <tr> <td><b>C</b></td> <td>ELASTIC BAND</td> <td>18CM</td> </tr> <tr> <td><b>D</b></td> <td>ADJUSTMENT</td> <td>8CM</td> </tr> <tr> <td><b>E/F/G</b></td> <td>PLEATS</td> <td>1CM</td> </tr> <tr> <td><b>H</b></td> <td>HEIGHT</td> <td>22CM</td> </tr> <tr> <td><b>I</b></td> <td>WIDTH</td> <td>19CM</td> </tr> </tbody> </table>	<b>A</b>	WIDTH	17CM	<b>B</b>	HEIGHT	10CM	<b>C</b>	ELASTIC BAND	18CM	<b>D</b>	ADJUSTMENT	8CM	<b>E/F/G</b>	PLEATS	1CM	<b>H</b>	HEIGHT	22CM	<b>I</b>	WIDTH	19CM
<b>A</b>	WIDTH	17CM																				
<b>B</b>	HEIGHT	10CM																				
<b>C</b>	ELASTIC BAND	18CM																				
<b>D</b>	ADJUSTMENT	8CM																				
<b>E/F/G</b>	PLEATS	1CM																				
<b>H</b>	HEIGHT	22CM																				
<b>I</b>	WIDTH	19CM																				
<p><b>Instructions for Use</b></p>	<p style="text-align: center;"><b><u>MASK PLACEMENT</u></b></p>  <ol style="list-style-type: none"> <li>1. Wash your hands with soap and water or an alcohol-based solution (<b>before</b> putting on the mask);</li> <li>2. Place the mask in the correct position. The folding edge must be faced up;</li> <li>3. Hold the mask by the supports / elastic bands and adapt to each ear;</li> <li>4. Adjust the mask next to the nose and chin, without touching the face of the mask.</li> </ol> <p style="text-align: center;"><b><u>MASK REMOVAL</u></b></p>  <ol style="list-style-type: none"> <li>1. Wash your hands with soap and water or an alcohol-based solution;</li> <li>2. Remove the mask just by holding the supports / elastic bands;</li> <li>3. Keep the mask away from the face and clothing, to avoid touching potentially contaminated surfaces of the mask;</li> <li>4. Place the mask in the appropriate container and sanitize your hands once again.</li> </ol>																					

**3. PACKAGING**

Product sold by the unit.

Note: These specifications are true and accurate according to the data provided by our supplier.

A: Armazém E, Sector X (Norte), Zona Industrial da Maia I, 4475-249 Maia – PORTUGAL T: 229 952 036 F: 229 969 331 M: geral@cpc.com.pt W: cpc.com.pt